



Perfectemp10

Temporary Crown & Bridge Material

The next generation of Perfectemp is everything that defines a premium multifunctional acrylic composite temporary crown and bridge material with exceptional, natural-looking esthetics. It delivers a high compressive and flexural strength and produces a minimal oxygen inhibited layer for optimal handling.

Perfectemp10 was created for the clinician who wants beautiful looking temporaries that are as strong and durable as leading competitors,* at a more affordable price.



- 10:1 automix formulation
- Superior strength
- Natural-looking luster
- Cartridge or syringe
- Five Vita shades
- 4:30 minute set time, 1:30 – 2:30 minute removal time

“I love Perfectemp10! The material is strong and looks natural. Perfectemp10 shines up nicely after polishing!”


— Tiffine Blanks, CDA

Perfectemp10 Technique




1

Take an impression of treatment area

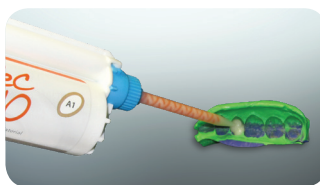

2

Extrude Perfectemp10 into impression

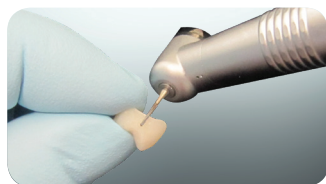

3

Trim, contour and finish temporary



4

Cement temporary into place

Available through

 **HENRY SCHEIN®**
DENTAL



To Order: 1-800-372-4346 8:00am–8:30pm (et)
www.henryscheindental.com



Features	Benefits
Uniform level structure	Produces high luster without polishing and hinders the accumulation of plaque
Only 3.4% shrinkage	Accurate fit and marginal integrity
Multiple uses	High esthetic temporary crowns, partial crowns, bridges, inlays, onlays and veneers
Minimal heat generation	Does not pose any threat to pulp
Minimal polishing required	Nice translucency — naturally beautiful temporaries
Free of Bisphenol A	No health effects on the treating dentist or patient by exposure to Bisphenol A

Physical Properties	
Compressive strength	400 MPa
Diametral tensile strength	70 MPa
Volume shrinkage	<3.4%

“As an added bonus, Perfecttemp10 requires no polishing or glazing to achieve a natural surface texture and dentition integration. It saves me time and creates exceptionally strong and beautiful temporaries.”
— Simona Cuevas, DDS

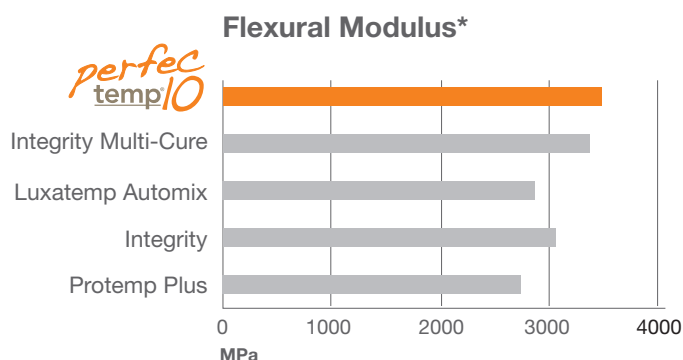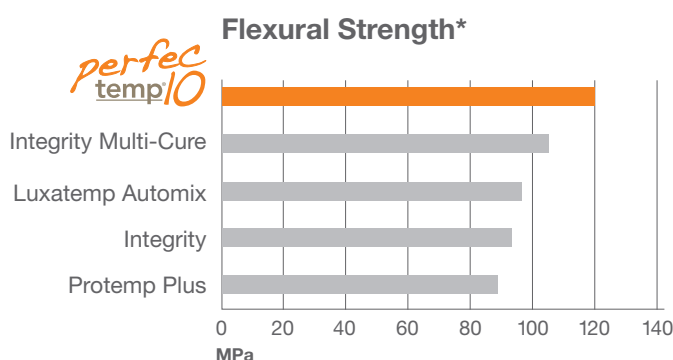

Product	Item #
Perfecttemp10 Syringe – 10 ml	
A1	CR1218
A2	CR1220
A3.5	CR1219
Contents: 1 – 10 ml syringe, mixing tips, instructions	
Perfecttemp10 Cartridge – 50 ml	
A1	SPD1305
A2	SPD1306
A3.5	SPD1307
B1	SPD1308
BL-2	SPD1309
Contents: 1 – 50 ml cartridge, mixing tips, instructions	

Available through



To Order: 1-800-372-4346 8:00am–8:30pm (et)
www.henryscheindental.com

*Data on file.