LumiSmile White in-office whitening is designed to comfortably and quickly whiten patients' teeth. The combination of LumiSmile White desensitizing enhancer and LumiSmile White whitening gel reduces chairtime required for effective whitening compared to traditional chairside bleaching treatments. This kit contains enough material to whiten both the upper and lower arches of one patient.

The LumiSmile White in-office whitening kit is for in-office use only.

Store at or below 75°. Do Not freeze.

**WARNINGS:**

- Product contains both strong oxidizing and alkaline agents and can cause chemical burns. Wear protective gloves and eyewear during preparation and application.
- Avoid contacting soft gingival tissue with both the desensitizing enhancer and whitening gel. Using the Paint-On Dental Dam included in the kit will shield gingival tissue from these materials. Vacuum LumiSmile White whitening gel off teeth as instructed before rinsing with water to prevent splattering. Should gel come in contact with soft tissues, rinse thoroughly to dilute.
- Material is not for injection.
- Keep out of the reach of children.

**INSTRUCTIONS FOR USE:**

1. Prophyl teeth to remove calculus and extrinsic stains before beginning the whitening treatment.
2. Insert cheek retractors. Have the patient bite down and rest tongue on a tongue block.
3. Take "before" photographs and record tooth color using a shade guide arranged in bleaching order.
4. Air-dry the gingival tissue and teeth.
5. Using the dispensing tip, apply Paint-On Dental Dam to the gingiva to isolate gums around the selected teeth. Overlap the dental dam onto the gingival tissue of adjacent teeth (about 0.5 mm) and light-cure for 1–2 seconds per tip width with a Sapphire® Plus Plasma Arc Curing (PAC) light.
6. Once the Paint-On Dental Dam is cured, apply Lip Moisturizer provided in the kit to the lips and mucosa, applying well beyond the vermilion border.
7. Prepare the auto-mix syringe.
  - Align the wedge-shaped key on edge of the static mixer housing with the opening in the syringe flange.
  - Push the static mixing tip onto the syringe and turn 90° clockwise until it stops.
8. Dispense a portion of the LumiSmile White whitening gel into separate dappen dish. Use the brush applicator to apply a 1–1.5 mm layer onto the labial surface of the teeth.
9. Install a sanitized Sapphire Whitening Crystal in place of the curing tip on the end of the Sapphire Plus PAC light pistol.

STEP 3

STEP 5

STEP 8

STEP 10

STEP 12

STEP 16

STEP 18

STEP 19

**BLEACHING WITHOUT THE SAPHIRE® PLUS PAC LIGHT**

Follow instructions, disregarding use of the Sapphire whitening Crystal; maintain gel on teeth as indicated, for the periods described. However, for maximum results, use of the Sapphire PAC light with whitening Crystal is recommended.

10. Position the Sapphire Whitening Crystal close to and directly in front of the teeth, using a combination of the articulating arm and chair adjustments. Center the Sapphire Whitening Crystal 1/4 to 1/2 inch from the teeth within the cheek retractor with the outer edges touching (if possible).
11. Press the Optional button on the Sapphire Plus PAC light keypad, then press 2 to select bleaching mode. The letters bL (bleach) will appear in the Program window, the number 60 will appear in the seconds window and 60 minutes will appear on the pistol handle.
12. Press the Start or pistol activator button to start the bleaching process. The number 60 (minutes) appears in the Program window and seconds begin to count down in the Seconds window.
13. When the Sapphire light beeps, fifteen (15) minutes have elapsed. Pause the light. Suction off whitening gel. Check on the comfort of the patient and reapply LumiSmile White whitening gel.
14. Unpause Sapphire curing light. After the second beep, 30 min have elapsed.
15. Move the Sapphire Whitening Crystal away from the teeth. Leave the Paint-On Dental Dam in place and vacuum gel from the teeth. Wipe with gauze and rinse the teeth while suctioning. Avoid splatter. Some patients with heavy discoloration or areas of unseen hypocalcification may need another 15 to 30-minute session to achieve desired whitening results. Evaluate the results with the patient, if possible.
 

*Note: In some cases you may have attained acceptable whitening in the first 30-minute session. If so, a second session is not indicated.*

*Note: In some cases, it may be necessary to peel away the Paint-On Dental Dam to make a shade determination. If the Paint-On Dental Dam is removed and another 15 to 30-minute application is indicated, remember to reapply the Paint-On Dental Dam to protect the gingival tissue.*
16. If another 15 to 30-minute session is desired, reapply the whitening gel following instructions, beginning at Step 7.
 

*Note: Be sure to inspect the Paint-On Dental Dam for any cracking or lifting before re-applying the desensitizing enhancer and whitening gel. If the seal has broken from the tissue, it is recommended to remove and replace the Paint-On Dental Dam and begin with Step 4.*
17. At the end of the second session, vacuum gel from the teeth, wipe away with gauze and rinse the teeth while suctioning. Avoid splatter.
18. Use a shade guide arranged in bleaching order to compare the color of the whitened teeth to the original shade recorded in Step 2 and take "after" photographs.
19. If patient is experiencing sensitivity, apply desensitizer gel for 5-30 minutes, as needed. Remove the tip from the end of the LumiSmile White desensitizing enhancer recappable syringe and dispense a portion of the LumiSmile White desensitizing enhancer into a dappen dish. Apply in a very thin (about 0.2 mm) layer on the labial surface of the teeth using the brush applicator provided in the kit and allow to sit for one minute. Wipe off any excess. Firmly recap the LumiSmile White desensitizing enhancer syringe. Send home remaining if needed.

To maintain desired shade, supply the patient with LumiSmile White Take-Home whitening kit for reapplication and touch-ups every few months. **Call today for more information and special pricing on LumiSmile White professional Take-Home whitening.**

**SUPPLEMENTAL PRODUCTS**

**ITEM NUMBER**

LumiSmile White Take-Home whitening Kit 16% Mint . . . . .	WHT1016
LumiSmile White Take-Home whitening Kit 22% Mint . . . . .	WHT1022
LumiSmile White Take-Home whitening Kit 32% Mint . . . . .	WHT1032
LumiSmile White Take-Home whitening Kit 16% Mint Refill . . . . .	WHT1116
LumiSmile White Take-Home whitening Kit 22% Mint Refill . . . . .	WHT1122
LumiSmile White Take-Home whitening Kit 32% Mint Refill . . . . .	WHT1132
LumiSmile White In-Office & 22% Take-Home whitening Combo Kit . . . . .	WHT0030

SDS SHEETS ARE AVAILABLE AT [www.denmat.com](http://www.denmat.com)

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