PATIENT INSTRUCTIONS

1. Floss teeth and then brush gently. (Fig. 1)
2. Apply a small amount of gel into the lower facial part of your custom-designed whitening trays (Fig. 2). Try to avoid dispensing an excessive amount into the tray. Using a cotton swab, remove any excess gel from the tray margins (Fig. 3). (If you are sensitive to peroxide, before inserting the trays, apply a thin amount of petroleum jelly to the gums adjacent to the teeth that will be whitened.)
3. Insert the whitening trays into the mouth (Fig. 4).
4. **For 16% formulation**, the recommended bleaching regimen is 1 treatment per day for 4–8 hours for 14 days or until desired results are achieved.
For 22% formulation, the recommended bleaching regimen is 1–2 treatments per day for 1–2 hours for 14 days, or until desired results are achieved.
For 32% formulation, the recommended bleaching regimen is 1–2 treatments per day for 10–15 minutes for 14 days, or until desired results are achieved.
5. Clean mouthguard thoroughly after each use and then brush your teeth again.
6. Rinse with an alcohol-free mouthwash.
7. **When you are ready for refills, contact your dental professional for additional LumiSmile White Take-Home whitening gel.**

Warnings: Keep out of the reach of children. Not for injection. If irritation (such as redness, swelling, soreness) of the gums or the mouth occurs, discontinue use and consult a dentist. Products containing peroxides are not recommended for use by children under 12 years of age. Use of product for periods of longer than 14 days is to be only under the supervision of a dentist. Avoid swallowing and avoid contact of the product with the eye.

Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow (Fig. 5).

Storage: Store at or below 75° F (23° C). Do not freeze

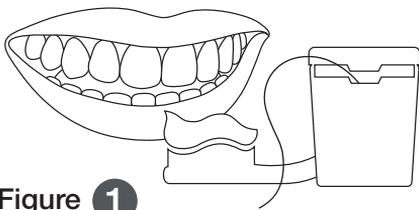
Figure 1

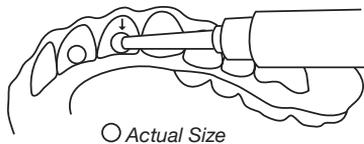
Figure 2

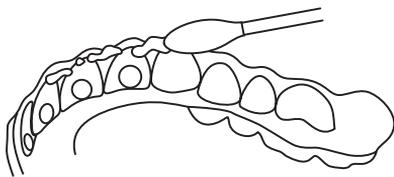
Figure 3

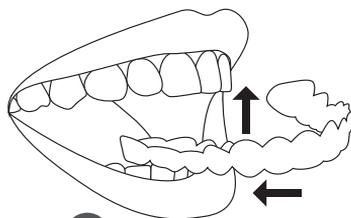
Figure 4

Make sure the gel does NOT touch the gums.
NO GEL IN THE RED ZONE!

Figure 5

IMPORTANT REMINDER:

Overloading trays with gel may cause temporary gum irritation. To avoid this, remove excess gel from the tray margins before placing the tray into your mouth. If this is an issue, reduce the amount of gel in the tray, and/or reduce wear time. If severe pain in discoloration of teeth or gums occurs, see your dentist.

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